



Treatment Planning System Guides

Eclipse™

Import

1. Choose File, Import, and then Wizard....
2. Choose DICOM Media File Import Filter
3. Select the directory corresponding to the Data Set using the Change Directory... button then select the DICOM Images and structure set from the list and click Next.
4. Click New
5. You may fill in demographic information or click the OK button
6. Click Next

You should now see the patient volume and structures...You are now ready to begin planning

Export

1. Select File, Export then Wizard
2. Select Plan then click next
3. Select Total Plan Dose then click Next
4. Click Next again
5. Select File Media Export Filter, then click Next
6. Click Yes
7. Click Change for All Objects. Browse to a directory then click the Finish button.

You are now ready to upload the Plan and Dose file to ProKnowsystems.com

NOTE: For additional questions about DICOM file import to, or export from, Eclipse, or if you're not seeing the features explained here, then please contact Eclipse support directly.