



Treatment Planning System Guides

Monaco™

Import

1. Click the Import New Data button on the Patient Selection dialog box
2. Click the Browse button to search your network for the DICOM patient you want to import.
3. Select the loaded patient from the DICOM Patient drop-down list.
4. Left click on the data you would like to import.
5. Select a local installation and clinic where you want this imported data to reside.
6. Monaco automatically shows a patient ID and Patient Name based on the DICOM Header information, or you can enter a new patient ID and Name.
7. Click the Add button to add the selected data to the import.

You should now see the patient volume and structures...You are now ready to begin planning

Export

1. Click Output tab | DICOM Export to show the DICOM Export dialog box.
2. On the left select RT Plan and Total Dose
3. On the right, select a destination where you can find the Plan and Dose file easily

You are now ready to upload the Plan and Dose file to [ProKnowsystems.com](https://www.proknowsystems.com)

NOTE: For additional questions about DICOM file import to, or export from, Monaco, or if you're not seeing the features explained here, then please contact Elekta support directly.