



---

## Treatment Planning System Guides

---

# Pinnacle™

### Import

1. The user may have to use a FTP client to transfer DICOM files from the Windows computer to Pinnacle and from Pinnacle back to Windows.
2. The DICOM files must be transferred to /autoDataSets/DICOM
3. Import the CT Images using Import CT
4. Once imported, create a plan then import the Structure set

**You are now ready to begin planning.**

### Export

1. Select File then select Export
2. Unless there is a preferred method, choose for the Destination AE Title: ADACRTP\_SCP
3. Select to transmit the RT Plan and RT Dose
4. A box may appear stating “The Plan is not locked yet. Ok to Continue?” choose Yes
5. The files will be transmitted to the folder /autoDataSets/DICOM or to /files/network/DICOM
6. Use your FTP client to transfer the RT Plan and RT Dose files from Pinnacle to the Windows computer

**You are now ready to upload the RT Plan and RT Dose file to ProKnowsystems.com**

NOTE: For additional questions about DICOM file import to, or export from, Pinnacle, or if you're not seeing the features explained here, then please contact Philips Pinnacle support directly.